

Tabletop Class B Steam Sterilizer, with European Standard En13060



- Bowie-Dick-Simulation Test (BDS)
- Helix Test
- Vacuum Test
- Prion



Chamber view



Trays



Water Tank



Drainage Port




LCD Display



USB & Printer



Sterilization Data

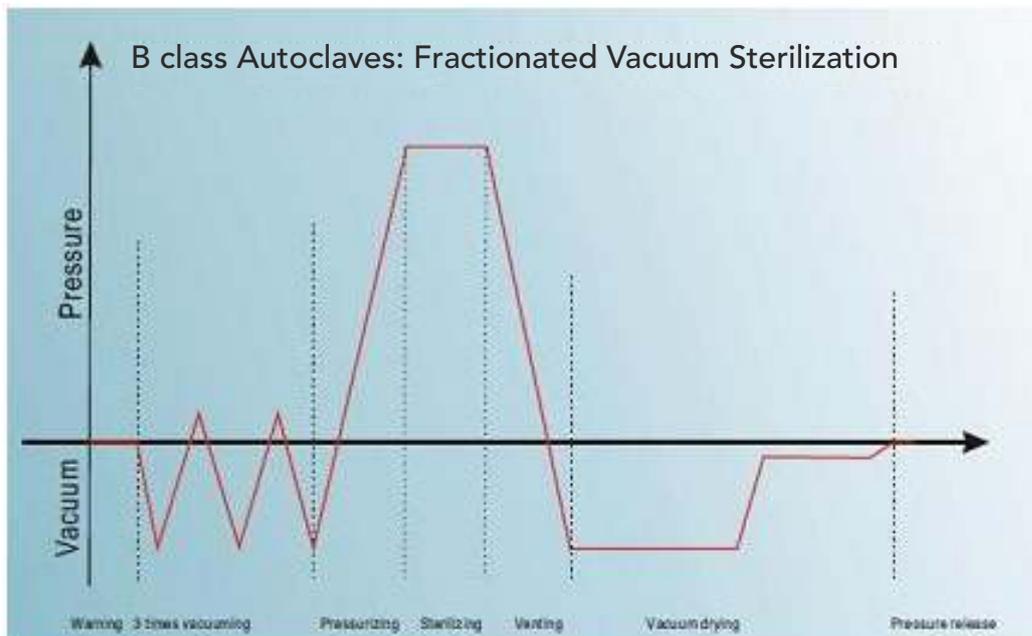

Technical Parameters:

- Chamber Size: Ø247×450mm Power: 2000W With 2 pcs vacuum pumps, more fast sterilization speed!!!
Have Liquid Program

Feature:

- High precision control system for more perfect sterilization.
- More fast sterilization time.
- Friendly and intuitive user interface with LCD screen, displays the information about cycle process by a large graphic all the time.
- A built-in independent steam generator, making the sterilization fast and completely.
- It works according to the post and pre-vacuum method with 2 vacuum pumps.
- Sterilizing time and Drying time can be adjusted.
- Multi-units of temperature and pressure can be chosen.
- Separated drying programs.(optional)

Safety:

- Door protecting locking system prevent the cycle from starting unless the door is properly locked and also prevents the door from opening if there have pressure in the chamber.
- Pressure safety valves prevent over pressurization in chamber and generator.
- Able to detect and identify the exact cause of any problem by warning and showing a specific error code, and also can output the exactly temperature and pressure at the time when error code appears.
- Water level switch in the water tank, controlling the max and min water level.

Documentation:

- Have printer built-in, which can print the sterilization records very easily. (Optional)
- Have USB port and USB driver which can save the records easily.
- Able to save at least 20 cycles of sterilization records in the internal memory, which can print or output at any time.

Program:

Program		Pressure(bar)	Sterilization Time (min)	Total Time (min)
Solid 134	134	2.1	4	18-25
Solid 121	121	1.1	20	30-38
Wrapped 134	134	2.1	6	30-35
Wrapped 121	121	1.1	20	35-45
Liquid 134	134	2.1	3.5	45-55
Liquid 121	121	1.1	3.5	55-70
Prion	134	2.1	18	50-60
B&D Test	134	2.1	3.5	22-30
Helix Test	134	2.1	3.5	22-30
Vacuum Test	----	----		15-20

Sales & Service in India by:



No. 12, 1st 'L' Main, 2nd Stage, 1st Block, Nagarabhavi, Bangalore - 560 072.
Phone: 080-23281009, Mobile: +91-94484 68510, E-mail: bv_helpline@hotmail.com